

Don't judge a mattress by its cover




Calm surface



- Inspect the exterior surface of each mattress cover for signs of damage, such as holes or cuts.
- Remove the cover and inspect its inside surface and the mattress core for staining or contamination.
- Safely dispose of any covers showing signs of damage or staining
- Arrange for contaminated mattress cores to be cleaned and decontaminated in accordance with the manufacturer's instructions or safely disposed of.
- Ensure that a frequent inspection regime is established for all mattresses.

What lies beneath

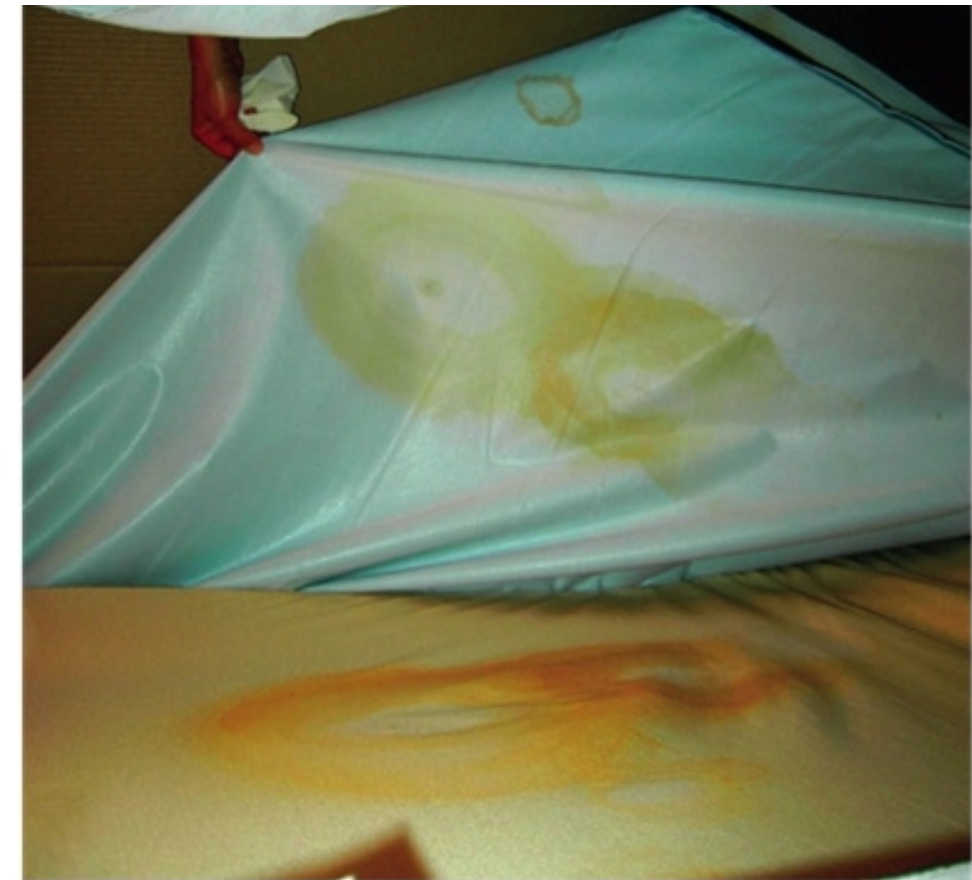

Do the user instructions for the mattress properly describe how to inspect, clean and decontaminate it? If not, tell us about it by reporting it as an adverse incident.

**Report adverse incidents online www.mhra.gov.uk
or on email aic@mhra.gsi.gov.uk
or telephone 020 3080 7080**

Medicines and Healthcare products Regulatory Agency
An executive agency of the Department of Health